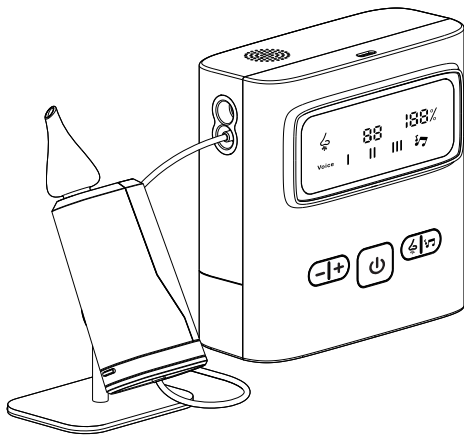
Hommie Nasal Aspirator

User Manual

Model: BC-024

EN (01-09) DE (10-18) FR (19-27) ES (28-36) IT (37-45)

IMPORTANT INSTRUCTIONS BEFORE USE

This device is designed to clear nasal secretions and mucus, and is suitable for children to adults. Please read this user manual carefully before use.

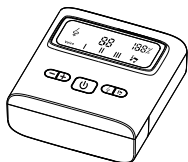
1. This device must be operated by an adult. Do not allow children to use it unattended.
2. Keep the product out of reach of children when not in use.
3. This device is intended for simple nasal cleaning only. If you or your kid experiences nasal discomfort, consult a healthcare professional.
4. When using this device to clear nasal secretions, ensure that the user (child or adult) breathes normally. Avoid holding breath, closing the mouth, or swallowing to prevent pressure imbalances in the eardrum caused by the vacuum.
5. Clean the device thoroughly after each use.

CAUTIONS

Use this product only as intended and with manufacturer-recommended attachments.

1. Avoid exposing the device to extreme temperatures, high humidity, or direct sunlight.
2. Do not place or store the device where it can fall or be pulled into a tub or sink.
3. Do not insert any objects into the device's openings or hoses.
4. Keep air openings clear of blockages, lint, hair, and soft surfaces like beds or couches.
5. Do not use while drowsy or sleeping.
6. Do not use if there is nasal bleeding, acute head/neck injury, coagulopathy, or bleeding disorders.
7. Insert the tip only enough to gently seal the nostril edge.
8. Stop using immediately if irritation occurs.
9. Do not use if the device is overheating, malfunctioning, or showing signs of damage.

PACKING LIST

Nasal Aspiration host
*1

Sponge
*5

Clean Brush
*1

Clip
*1

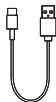
Tube
*1

Tip
*2


Mucus Collector
*1


USB-C Cable
*1

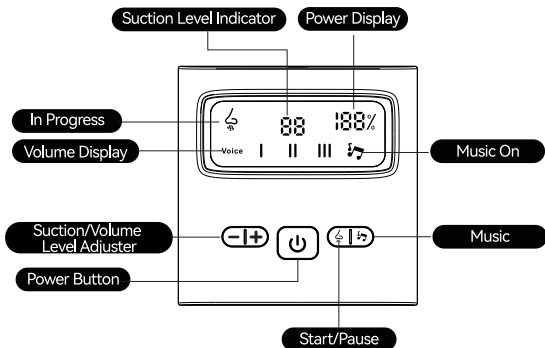Product Overview

ON/OFF	Long press "Power Button" for 3s to turn on/off the nasal aspirator.
Start/Pause	Press "Start/Pause" once to start or pause suction.
Suction/Volume Level Adjuster	Press "+" to increase suction or volume. Press "-" to decrease suction or volume.
Power Display	Real-time power display.
Music	Press "Music" to play music.
Night Light	Press "Night Light" to turn on the night light.

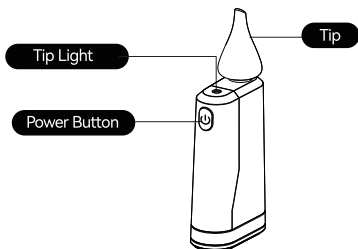Mucus Collector


Product Specification

Vacuum Suction Pressure	225-412mmHg (65±5kPa), adjustable in 9 levels
Noise Level	45-55db
Operating Temperature	5-35°C
Battery Capacity	1500mAh
Battery Life	Up to 2.5H on a full charge
Charging Voltage	5V
Input Current	1A
Power Source	USB-C

Operating Instructions

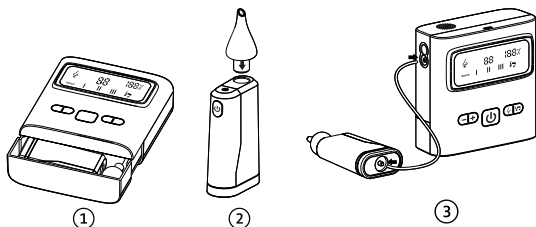
First Assembly After Arrival


- ① Remove the snot container, tubing, and appropriate tip from the storage drawer of the host.
- ② Attach the silicone tip to the snot container.
- ③ Connect the tubing to both the host and snot container, now power on the device and start suction.

How to Remove and Wash

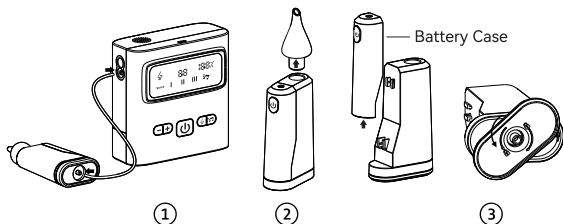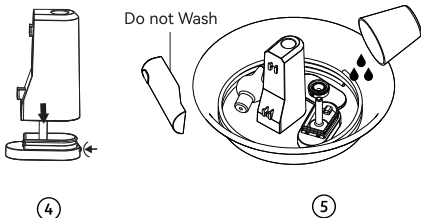※ The sealing plate is not washable.

① Disconnect Tubing

Unplug the tubing from the device and the snot container.

② Remove Tip and Battery Box

- Detach the silicone tip from the snot container.
- Slide up and remove the battery box.

③ Remove Snot Container Bottom Cover

- Turn the snot container upside down.
- Pinch the bottom cover with your thumb and index finger, then twist it slightly to the left (about 80°) to unlock and remove.

Note: Do not wash this part unless mucus has entered the sponge.

④ Remove Transitional Component

- Pinch the elastic buttons on both sides of the base with your thumb and index finger.
- Gently pull out the transitional component.

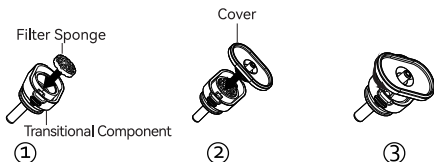
⑤ Clean Components

- Rinse the snot container and tip with water.
- Use a brush to scrub if necessary.
- Air dry and place the components back into the device's storage drawer.

Note: The tubing does not need to be washed unless stained.

Special Notes on Sponges

The sponge generally does not need cleaning or replacement. However, if it becomes contaminated with mucus, you will need to remove and replace it. Please follow these steps:


- ① Remove or place the filter sponge on the transitional component as shown in Picture 1, then loosen or fasten the cover as shown in Picture 2.
- ② Rotate the cover counterclockwise to unlock or clockwise to lock, as shown in Picture 3.

Note: This sponge is not disposable and cannot be washed, dried, and reused.

How to Assemble the Snot Container After Cleaning

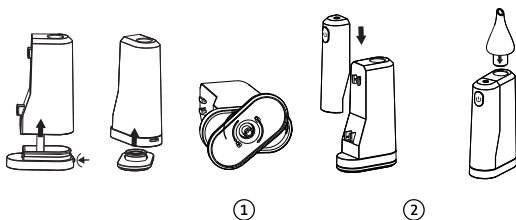- ① Align the buckle on the sealing plate with the bayonet on the base. Press it in and turn it to the right to lock.
- ② Dock the base with the snot container and push the base into the container until you hear a “click,” confirming it is securely installed.

Troubleshooting

Weak or No Suction

- ① Ensure the silicone suction tip is installed correctly.
- ② Charge the device before use.
- ③ Check that all components are assembled properly.
- ④ Inspect the sponge on the sealing plate for moisture or expansion; if present, replace it (multiple replacements are included).

Injury or Pain

- ① Do not use if you have nosebleeds or acute injuries to the head, face, or neck.
- ② Avoid inserting the silicone suction tip too deeply into the nasal cavity.
- ③ Do not use continuously for long periods or at an excessively high suction level.
- ④ For severe nasal congestion or uncertain conditions, consult a doctor.

No Sniffing or Minimal Sniffing

- ① Position the suction tip correctly in the nasal cavity and keep the head upright.
- ② Choose the appropriate suction tip based on the viscosity of the nasal mucus.
- ③ If needed, spray saline into the nasal cavity to dilute the mucus.

Kind Reminder

- ① Use only mild soap and water for cleaning.
- ② After cleaning, the snot container (remove base and sealing plate) and silicone tips can be sterilized with boiling water.

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Manufacturer: Dongguanshi Yiyangmei Technology Co.,Ltd.
EC | REP Ocean Trading GmbH
UK | UKRP OCEAN SUPPORT LTD